




# LONG-LOOP EASI-GRIP SCISSORS


## CODE

- Right Hand with short round ended blade - LEG-1
- Right Hand with short pointed blade - LEG-3
- Right Hand with long pointed blade - LEG-5

## DESCRIPTION

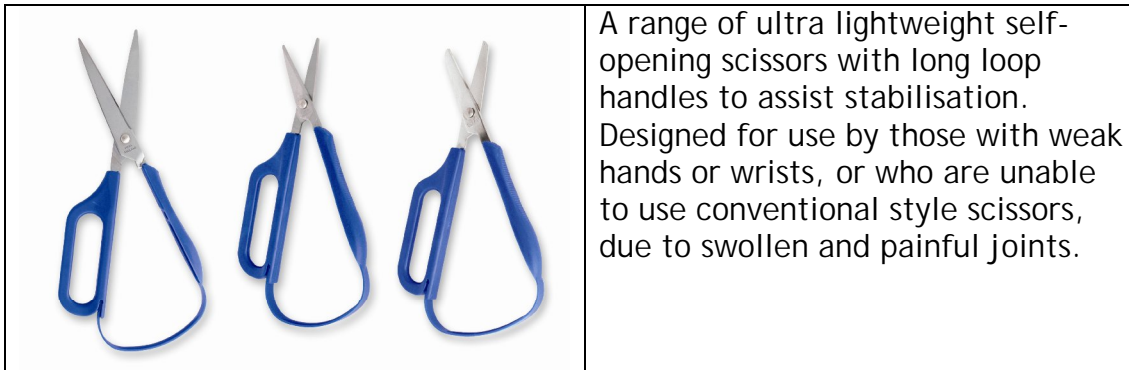

## HOW TO USE

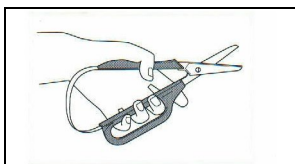

Using the index finger to provide guidance and control, the middle, ring and little fingers are inserted through the loop to provide the strength for the cutting action. Alternatively for those with very limited movement the

fingers can be placed over the long loop. Gently squeeze the scissors shut, the spring-action of the handle will automatically reopen the scissors as soon as the grip is eased.

## KEY FEATURES

- High quality handle that will not snap and retains spring action
- Stainless steel blades with guaranteed secure fixing to handle
- Colour coded handles easily identify whether for right hand (blue), or for left hand (green)
- Blade guard supplied for safety
- Ultra lightweight

## RECOMMENDED FOR

Weak Grip	Poor Hand Control



## RELATED PRODUCTS

- LEG-1/L - Left Hand Easi-Grip® with short round ended blade
- LEG-5/L - Left Hand Easi-Grip® with long pointed blade

### Peta (UK) Ltd

Mark's Hall, Mark's Hall Lane, Margaret Roding, Dunmow, CM6 1QT, England

Tel : +44 (0) 1245 231118 Fax : +44 (0) 1245 231811

E-mail : sales@peta-uk.com Internet : peta-uk.com