


## DESCRIPTION

	<p>These scissors are easy to use simply requiring a pushing down action on the "T" shaped handle. The spring will automatically reopen the scissors once the pressure is released.</p> <p>Ideal for children or adults either with poor strength or who are unable to use scissors with conventional loops.</p>
---	--

## HOW TO USE

Place the scissors on the table with one "T" handle flat on the surface. Pushing down on the top "T" handle closes the blades. A discreet spring will automatically reopen the scissors ready for the next cut. The scissors can be used in two ways: By maintaining the paper in fixed position it can be cut by "scooting" the scissors along the table; alternatively you can keep the scissors still (placing it on a Dycem<sup>®</sup> mat may assist with stability) and moving the paper towards the scissors.

## KEY FEATURES

- Large "T" shaped handles to assist grip and leverage
- Stainless steel blades with guaranteed secure fixing to handles
- Discreet spring to automatically reopen the scissors
- Blade guard supplied for safety
- Ultra lightweight

## RECOMMENDED FOR

		
Weak Grip	Poor Hand Control	Tremor



## RELATED PRODUCTS

For greater stability plastic and wooden based tabletop scissors are available, ask for details of PTT-1/PB and PTT-1/WB

### Peta (UK) Ltd

Mark's Hall, Mark's Hall Lane, Margaret Roding, Dunmow, CM6 1QT, England

Tel : +44 (0) 1245 231118 Fax : +44 (0) 1245 231811

E-mail : sales@peta-uk.com Internet : peta-uk.com