




# DUAL CONTROL TRAINING SCISSORS


## CODE

- Right Hand with short round ended blade - PTR-1
- Left Hand with short round ended blade - PTL-1
- Right Hand with long pointed blade - PTR-5
- Left Hand with long pointed blade - PTL-5

## DESCRIPTION

	<p>Developed, under consultation with Occupational Therapists, to assist the client who:</p> <ul style="list-style-type: none"> <li>- Has poor visuo motor co-ordination</li> <li>- Lacks the strength to complete a cutting action</li> <li>- Has a tremor and cannot place the scissors at a given starting point</li> <li>- Has poor background/ foreground differentiation</li> </ul> <p>Ideal for learning the opening / closing action of scissor skills</p>
---	--

## HOW TO USE

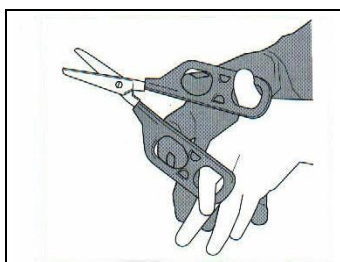

The client places his/her fingers through the loops furthest from the blade, whilst the therapist places their fingers over the client's hand and through the loops nearest the blade to assist with placement, guidance and strength for the cutting action.

## KEY FEATURES

- Double looped handles allows child and adult to simultaneously complete the cutting action
- Comfortable grip given by wide finger contact area
- Colour coded handles easily identify whether for right hand (blue), or for left hand (green)
- Stainless steel blades with guaranteed secure fixing to handles

## RECOMMENDED FOR

Weak Grip	Poor Hand Control	Visual Impairment	Tremor



### Peta (UK) Ltd

Charles House, Park Farm, Kelvedon Road, Inworth, Colchester, CO5 9SH, England  
 Tel : +44 (0) 1376 573476 Fax : +44 (0) 1376 570023  
 E-mail : sales@peta-uk.com Internet : peta-uk.com